FFP2 PARTICLE FILTERING HALF MASK

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Ear-loop
Maintenance-free
C-Shaped
SNN70369B FFP2 NR

USER INSTRUCTION

- ① Failure to follow all instructions and limitations could seriously reduce the effectiveness of this particle half mask and could lead to illness, injury, or death.
- ② A properly selected mask is essential. Before occupational use, the wearer must be trained by the employer in the correct use of the mask in accordance with applicable safety and health standards.
- ③ This particle half mask does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life.
- ④ Discard the mask and replace with a new one if:
 - Excessive clogging of the particle half mask cause breathing difficulty.
 - The particle half mask become damaged.
- ⑤ Leave the contaminated area if dizziness, irritation, or other distress occurs.

THE MAIN RAW MATERIALS

- Biomass Graphene spunbond non-woven fabric (PP)
- Melt-blown non-woven fabric (PP)
- Spunbond non-woven fabric (PP)

FITTING INSTRUCTION

- ① Place fingers inside the particle half mask. Bend the nose clip around fingers to form nose shape.
- ② Hold the particle half mask in position over the nose and mouth, pull the ear loop behind the ears.
- ③ Using both hands, Adjust the nose clip and make a fitting curve inside. Shift the ear loop to ensure a positive face seal.
- ④ Fit check:
 - To test the fit of the particle half mask without an exhalation valve, cup both hands over mask and inhale sharply.
 - If air flow is felt in the nose area, re-adjust/tighten the nose clip.
 - If air flow is felt around the edges of the mask, re-position the particle half mask / ear loop to achieve a better fit.
- ⑤ Change the mask immediately if breathing becomes difficult or mask becomes damaged or distorted.
- ⑥ Change the mask if a proper face seal cannot be achieved.
- ⑦ Recommended usage time: 4-8 hours.
- ⑧ Careful observance of these instruction is an important step in safe mask use.

USE LIMITATION

- ① Do not use the mask or enter or stay in a contaminated area under the following circumstance.
 - Atmosphere contains less than 19.5% oxygen.
 - If you smell or taste contaminant.
 - For protection against gases or vapors.
 - Contaminants or their concentrations are unknown or immediately dangerous to life or health.
 - For sandblasting, paint-spray operations and asbestos.
 - In explosive atmospheres.
- ② Do not modify or misuse the mask.
- ③ Do not use the particle half mask with facial hair or any other conditions that may prevent a good face-seal as the requirements for leakage will be achieved.
- ④ Particle filtering half mask needs to be inspected prior to each use to assure there are no holes in the breathing zone other than punctures around and staples and no damaged has occurred. Enlarged holes resulting from ripped or torn filter material around staple punctures are considered damage.
- ⑤ This mask helps protect against certain particulate contaminants but does not eliminate exposure to risk or contracting disease or infection. Misuse may result in sickness or death.
- ⑥ This particle half mask marked "NR", shall not be used for more than one shift.
- ⑦ This particle half mask is not suitable for users who have damaged ears.

40 pcs

Ear-Loop
Maintenance-free

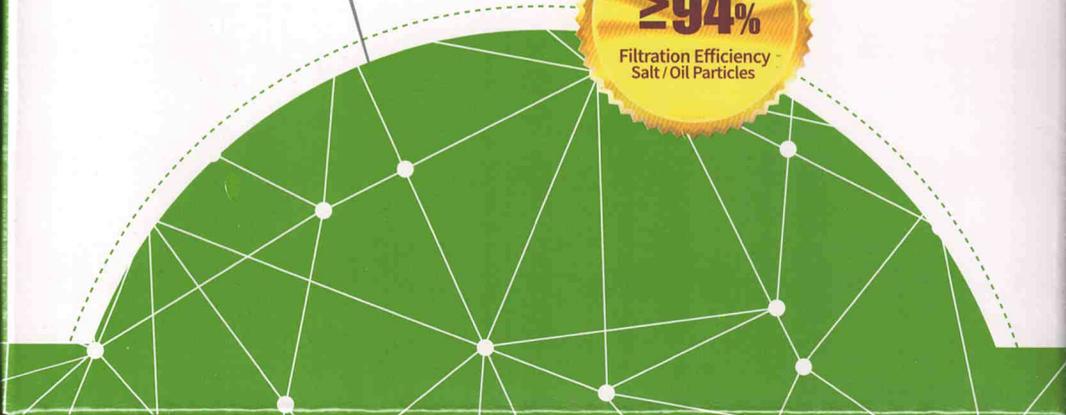
SHENGQUAN

Healfiber

FFP2 PARTICLE FILTERING HALF MASK

CE 2834 EN 149:2001+A1:2009 FFP2 NR

Temperature range of storage conditions: -20°C to +30°C

Maximum relative humidity of storage conditions: <80%

Shelf life: 2022/08
3 years from date of manufacture

See information supplied by the manufacturer

Product Code: SMN/70369B
Product Name: Particle filtering hair mask (C-Shaped)
Batch No.: H20909Y013
Production Date: 09.09.2020

MADE IN CHINA
Scan the QR code to check the product genuine or not. One box one security code. The quality can be traced.

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【Service Hotline】: 0086-400-777-8118

【Manufacturer】: Shandong Shengquan New
Materials Co., Ltd.

【Address】: Diaozhen Industrial Development
Zone, Zhangqiu District, Jinan, Shandong, China